# BufferPro™

8.4% Sodium Bicarbonate Buffer Solution

## BufferPro™

8.4% Sodium Bicarbonate Buffer Solution

EN

### INSTRUCTIONS FOR USE

#### Description:

BufferPro is a sterile solution of Sodium Bicarbonate in water used to neutralize dental local anesthetic cartridges prior to injection. The solution is within a capsule to be directly transferred into the anesthetic cartridge.

Each box contain 25 Single-Dose 0.1mL Capsules.

Each 0.1mL capsule contains 8.4mg of Sodium Bicarbonate in water for injection. pH 7.8 (7.0–8.5). Contains no bacteriostat.

#### Intended Use:

BufferPro is indicated for the in vitro buffering of commercial preparations of local dental anesthetics in 1.7/1.8mL cartridges, prior to administration, to adjust their pH to a more physiological value.

#### Contents:

Each capsule is supplied sterile and individually packaged for single use.

SKU	Contents
01B0000	Box of 25 - 0.1 mL Capsules

#### Contraindications:

- Do not use with non-dental local anesthetic cartridges.
- BufferPro is not indicated for direct line administration of solutions to any patient or user.

#### PRECAUTIONS:

- Inspect packaging prior to use. Do NOT use if sterile packaging barrier has been damaged or compromised.
- Use only as directed.
- Refer to the dental local anesthetic IFU for full list of anesthetic's contraindications, precautions, warnings, and dosages.
- Use BufferPro immediately prior to use of dental local anesthetic.

#### WARNINGS:

- Do NOT use if sterile packaging barrier has been damaged or compromised.
- Do not use past the date of expiration.
- Do not insert finger into the BufferPro Capsule to avoid potential needle stick injury.
- Single Use Only - Do not use one BufferPro capsule on multiple dental local anesthetic cartridges.
- Use only one BufferPro capsule on one dental local anesthetic cartridge.

#### Adverse Events:

No known adverse events are associated with BufferPro.


If a serious incident has occurred in relation to the device, please report the incident to the Manufacturer and/or competent authority of the Member State where the device was used.

#### Materials:

Sodium Bicarbonate, USP Sterile Water.

#### Directions for Use:

1. Remove the capsule from the sterile foil.
2. Insert the open end of the BufferPro capsule onto the front of the dental local anesthetic cartridge (this is the end with the aluminum cap and rubber diaphragm, as seen in Figure 1) and push the BufferPro capsule until it stops and all solution has been transferred (Figure 2).

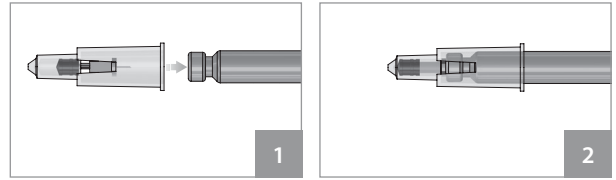**Note:** The rubber plunger at the back of the dental cartridge may move slightly backward as the sodium bicarbonate is transferred.

3. Remove the BufferPro capsule and dispose per disposal instructions.
4. To ensure thorough mixing of the sodium bicarbonate from BufferPro, invert the dental local anesthetic cartridge once or twice prior to use.
5. Use dental local anesthesia cartridge immediately after buffering.

#### Storage:

- Store at room temperature between 15°C (59°F) to 25°C (77°F).
- Do not freeze

#### Shelf Life:

- BufferPro has a shelf life of 2 years from the date of manufacturing.
- Do not use if past the date of expiration.

#### Disposal:

Dispose the used or contaminated BufferPro capsule within a sharps container in accordance to facility or institutional guidelines/procedures.

#### For Further Information:

To obtain SDS or IFU, visit [septodontusa.com](http://septodontusa.com) or call Septodont at 800-872-8305.

Symbols	Symbol Meaning
	Catalog number
	Batch number
	Expiration Date
	Sterilized using irradiation (Gamma)
	Caution: See instructions for use
	Do not use if package is damaged
	Single-use only
<b>Rx ONLY</b>	CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician
	Indicates the lower and upper limit of temperature to which the device can be safely exposed.
	See Instructions For Use on <a href="http://septodontusa.com">septodontusa.com</a>

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Plymouth Meeting, PA 19462 U.S.A.
Septodont Inc.
Lancaster, PA 17601, U.S.A.